

LASCOD

dental clinic use



Ghenesyl

addition curing polyvinylsiloxane silicone

Polyvinylsiloxane addition curing silicone is a versatile and efficient product guaranteeing maximum precision in humid conditions. Ghenesyl, now with its renewed formula, has different grades of hardness (Hard/Soft) and consistency (Superlight/Light/Regular/Heavy) which attain the best results adapting to every impression technique.

A long working time increases the product workability. At the same time a short setting time in mouth is guarantee for the best patient's comfort. Ghenesyl is now available in two different setting releases: normal and the all-new fast version which is faster in the working time and setting time in mouth.

New formula!



new

Fast Set

mint flavour

25µm

Ghenesyl Putty, Normal Set and Fast Set are available now with improved workability and a slight mint flavour.



PUTTY



Ghenesyl mono


monophase addition curing
polyvinylsiloxane silicone

Ghenesyl Mono, developed by LASCOD, is a new impression material with a medium consistency, ideal for monophase technique. Recommended for crowns and bridges impression taking, implantoprosthesis, inlay, onlay, partial and total edentulism. Thanks to its characteristic viscosity it can be used directly on the surfaces to be reproduced and on impression tray. It's syringable through the included mixing tips or through the specific syringe for elastomers. Given its optimal flexibility Ghenesyl Mono can be easily removed from the mouth without structural alteration.

Ghenesyl Mono can be used also with Super Light Body / Light Body / Regular Body silicones as support material inside individual impression tray or standard stainless steel impression tray.

- Ideal viscosity for monophase technique
- High details definition (5 μm)
- Maximum thixotropicity and hydrophilicity
- Fast setting time in mouth
- High resistance to tear
- High elastic recovery
- High dimensional stability over time
- Slight mint flavour
- Resistant to aseptic cleaners

Mint
Flavour

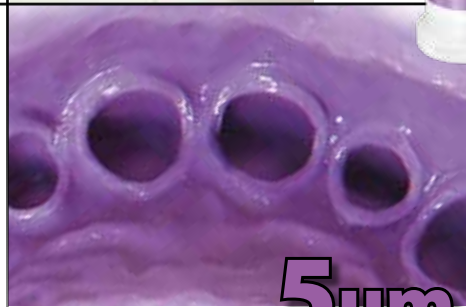

5 μm



Oklurest

silicone for
bite registration

Oklurest is an addition silicone (polyvinylsiloxane) made for orthodontic bite registration, registration keys for gnathologic registrations, inter-maxillary registration keys for centered positions, eruptions and ectopic eruptions, registration for cephalometric analysis, which requires a CAD scanning (optical, laser and tactile).

Oklurest characteristics grant a working base which allow the technicians to obtain the maximum precision.

The high dimensional stability over time ensures optimal conservation even in the presence of temperature changes.

imperceptible viscosity

Oklurest is easy-to-use and offers a series of advantages which allow the maximum control during positioning.

The viscosity is imperceptible for patient.




The useful hardness degree (95 Shore A) achieved in short time, allows to work on masses widely stable facilitating the finishing stages by cutters or burs.

Working time (23°C)	45"
Minimum time in mouth (37°C)	60"
Hardness Shore A	95
Linear dimensional change (after 24 hours)	-0,02%



- Easily and safely workable thanks to double cartridge system 1:1
- Maximum control during positioning
- Thixotropic
- Imperceptible viscosity for patient
- Exact reproduction of occlusal details (5µm)
- Short setting time
- Minimum dimensional variation
- Undeformable / stable over time
- Easy insertion and removal from the model
- Easy finishing by burs

maximum precision

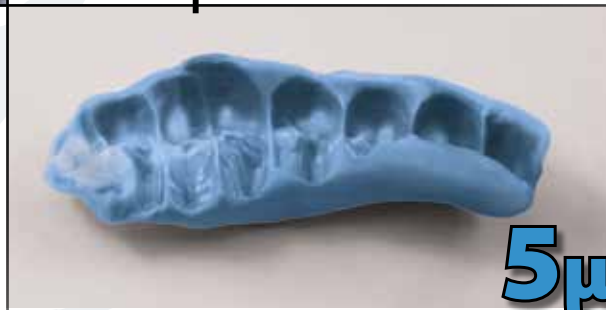

95 Shore A


CAD
SYSTEM

thixotropicity, accuracy & hydrophilicity



Ghenesyl, now with its renewed formula, is highly thixotropic and hydrophilic. The material stays in place after dispensing and it flows well under pressure. Ghenesyl easily adapts to the anatomical shape of dental structures and soft tissues, thus reproducing perfectly integral margins.


5µm

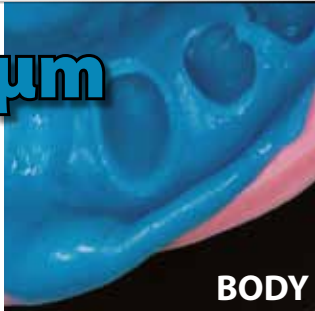

BODY



The contact angle measurement (wettability index) is less than 10°.

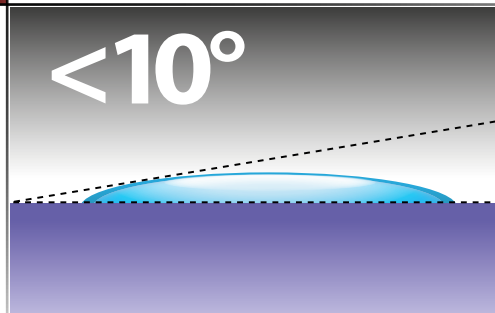

GHENESYL KIT

Ghenesyl is available in KIT PACKS containing 2x150ml putty base and catalyst and a cartridge of 50ml body.

	Working time including mixing time (23°)		Minimum time in mouth (37°)		Recovery from deformation %	Dimensional stability	PUTTY/ WASH	SANDWICH	MONOPHASE
	NORMAL SET	FAST SET	NORMAL SET	FAST SET					
PUTTY SOFT	2' 30"	1' 15"	2' 15"	1' 30"	99.9	7 days			
PUTTY HARD	2' 30"	1' 15"	2' 15"	1' 30"	99.9	7 days			
SUPERLIGHT	2'	1' 20"	1' 30"	60"	99.9	7 days			
LIGHT	2'	1' 20"	1' 30"	60"	99.9	7 days			
REGULAR	2'	1' 20"	1' 30"	60"	99.9	7 days			
HEAVY	2'	1' 20"	1' 30"	60"	99.9	7 days			
MONO	2'	-	1' 30"	-	99.9	7 days			

packaging

*** ALSO AVAILABLE
IN ECONOMY PACK**
4 x 50 ml cartridges
without mixing tips

	GHENESYL	NORMAL	FAST	
	PUTTY HARD	GNS020	GNS520	2 x 300 ml base + catalyst
	PUTTY SOFT	GNS010	GNS510	2 x 300 ml base + catalyst
	HEAVY BODY*	GNS070	GNS570	2 x 50 ml cartridges + 12 mixing tips
	SUPERLIGHT BODY*	GNS040	GNS540	2 x 50 ml cartridges + 12 mixing tips
	LIGHT BODY*	GNS050	GNS550	2 x 50 ml cartridges + 12 mixing tips
	REGULAR BODY*	GNS060	GNS560	2 x 50 ml cartridges + 12 mixing tips
	MONO* (50 ml)	GNS400	-	2 x 50 ml cartridges + 6 mixing tips
	MONO (380 ml)	GNS480	-	2 x 380 ml cartridges + 2 mixing tips
	KIT PUTTY HARD + LIGHT BODY	GNS080	GNS580	
	KIT PUTTY HARD + SUPERLIGHT BODY	GNS081	GNS581	2 x 150 ml (putty) + 1 x 50 ml cartridge (body) + 6 mixing tips + 6 oral tips
	KIT PUTTY SOFT + LIGHT BODY	GNS082	GNS582	
	KIT PUTTY SOFT + SUPERLIGHT BODY	GNS083	GNS583	
OKLUREST				
	<i>Silicone for bite registration *</i>	OKR095	-	2 x 50 ml cartridges + 12 mixing tips

SILICONE ACCESSORIES



Dispenser (50ml)
GNS100
60 oral tips
GNS300 - superlight/light
GNS350 - regular/heavy

48 mixing tips (50ml)
SIL002 - superlight/light
SIL003 - regular/mono
SIL004 - heavy - OkLurest

24 mixing tips (380ml)
GNS280 - mono

The photographic images here reproduced are purely indicative and are not necessarily identical to the actual products.

LASCOD

For further
information
activate the
QR Code reader
on your smartphone.



Follow us!

